OPED GmbH
Mühltal 3
D-83626 Valley
Germany

Tel. +49 (0)8020 / 9079-0
Fax +49 (0)8020 / 9079-99

mail@oped.de
www.oped.de

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english

With VACOankle, you get an ankle splint that ideally combines stability and wearing comfort. A self-adapting vacuum system combined with a stable lattice frame structure lets you walk normally with maximum protection given to your ankle's injured ligaments. What's more, the vacuum cushion allows optimal customised adaptation to all kinds of swelling conditions without causing any bruising.

Indications: Injuries to the ligaments of the ankle joint

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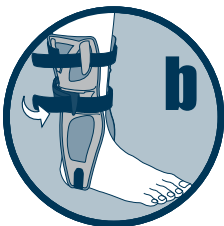
VACOankle anlegen Applying VACOankle

Application de VACOankle Colocación del VACOankle

1 Von hinten Anlegen Attach From Behind


- **a** Untere Kante der Schiene soll mit der Fußsohle abschließen
- **a** The lower edge of the splint plate should end at the sole of the foot
- **a** Fixer l'arête inférieure de l'attelle à la semelle du pied
- **a** El borde inferior de la ferula debe quedar al ras con la planta del pie

- **b** Mittleren Gurt fest schließen
- **b** Fasten the middle strap tightly.
- **b** Attachez la bande centrale
- **b** Cerrar firmemente la correa central

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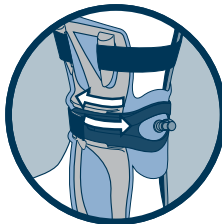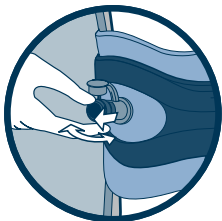
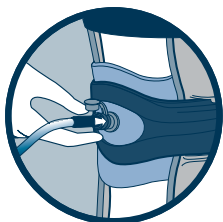
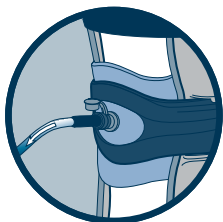
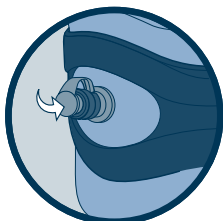
english

Safety tips

To ensure optimum support VACOankle should be worn inside a laced shoe. Thus, a more effective way of limiting the lateral bending potential of the foot can be accomplished without restricting the normal walking gait. Stability and comfort can be increased by evacuating the vacuum cushion (as described in chapter "Individual Fitting"), although a non-evacuated cushion offers sufficient support already.

Wearing a sock underneath is recommended as it improves comfort and hygiene.

Cleaning and Care

Wash your VACOankle by hand with lukewarm water and a mild detergent. Use a cloth to dry the splint or let it dry at room temperature.

Like all our products, VACOankle is recyclable.

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Service Informations

Latest information are available at: **www.oped.ch**
If you have further questions or wish to make a new order, please contact your retailer or get directly in touch with OPED at:

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OPED GmbH
Mühlthal 3
D-83626 Valley
Germany

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OPED AG
OPED Inc.