

Treatment week	1	2	3	4	5	6	7	8	9	10		
Ankle Fracture Weber A												
Ankle Fracture Weber B operativ												
Ankle Fracture Weber C/ Maisonneuve												
Pilon-Tibial							Load increase on the basis of radiological report					
Calcaneus operative							Load increase on the basis of radiological report					
Calcaneus conservative							Load increase on the basis of radiological report					
Metatarsal		VACOpedes 0° sole										
Jones Fraktur												
Hallux		VACOpedes decompression sole										
Arthrodesis Lower Ankle Joint										Load increase on the basis of radiological report		
Arthrodesis Upper Ankle Joint											Load increase on the basis of radiological report	
Achilles operative/conservative												
Fibular ligaments				VACOankle brace after 3 weeks								

Sole contact
 Partial load
 Full load

Storage VACOsplint

VACOped closed Immobilization with 0°, 15° or 30° Adapter (dorsal and ventral)

Physical therapy

VACOped closed release movement Upper Ankle Joint with 10°-0°-10° Adapter dorsal